Artwork Version:	
Repro Version:	STAGE: 3 VERSION: 1
Job No:	5738014
Cutter Ref:	
Job Title:	Orkla Care AB (Digital) SalveloxMed Leaflet Sloxmed Blister Rescue (51030097)
Design Ref:	DF 366676
Printer:	Schur Pack Denmark A S
Contact:	
Supplier:	Orkla Care AB (Digital)
Contact:	
SGS Contact:	Louise Blow (P)
Operator:	jasmine.read
Modify Date:	22-MAY-19 13:29:22
Design Barcode:	
MySGS Barcode:	TBC
Magnification:	
BWR:	0
Print Process:	Litho
Profile:	EO07 9900SM ECI Offset ISO Coated 39L
Substrate:	

PANTONE 2757 C 

Line -

6.96% - 2970mm²

Total Ink Coverage: 7.62%

Supplier/Printer Sign Off:

Please sign and date against your appropriate title after checking proof and indicate clearly any comment or corrections.

Supplier:	
Date:	
Comments:	
Printer:	
Date:	
Comments:	

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EN BLISTER RESCUE

Blister Rescue is intended for use on punctured or unpunctured blisters on feet, and also for blister prevention. The product provides a complete 360° Protective Care of the blister, with the following five functions: • Protects blistered skin. • Easy to apply. • Stays in place securely. • Easy and painfree to remove. • Discreet, transparent and adapts to skin colour.

INSTRUCTIONS

1. Clean and dry the skin from dirt and lotion before applying the plaster. Tear to open the protective plaster pouch.
 2. Remove the plaster's protective paper (marked 1).
 3. Apply the plaster centered over the blister and try to minimize wrinkles. Remove the white folded tab.
 4. Remove the protective plastic top layer (marked 2).
- You can keep the plaster on until it falls off, not more than five days. Change if the gel pad gets saturated.

WARNING:

Do not use on blood blisters or deep, heavily bleeding or infected blisters. People with impaired wound healing, e.g. diabetics/elderly people, should check their blister regularly. Consult a doctor if an infection occurs or if the blister has not healed within a week. Product to be used only once by one user. Re-use of the product increases the risk of infection.

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232mm

Content:	<input type="checkbox"/>	Signed:	
Colour:	<input type="checkbox"/>	Signed:	
Retoucher Check:	<input type="checkbox"/>	Signed:	
		Date:	